


HOW TO PROPERLY SCAN AN ANIMAL FOR A MICROCHIP

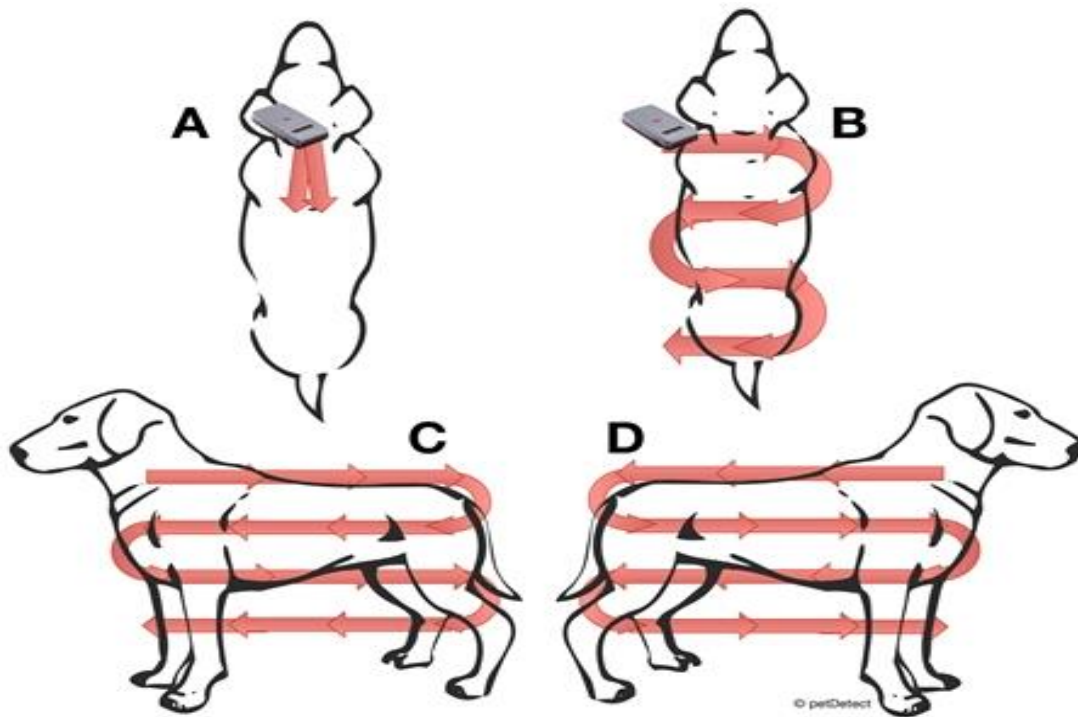

1. Turn on the scanner by pushing the power button on.
2. Once on, you may begin scanning.
3. Hold the scanner over the surface of the animal at all times when scanning; all scanners should be held in contact with the fur or skin.
4. The scanner should be moved in small circular motions over the animal; this action helps maximize the potential for microchip orientation and successful scanning.
5. All areas A,B,C, and D need to be scanned due to potential chip migration.
6. Scanner will vibrate if a chip is found and it will appear on the screen.
7. AAHA universal lookup tool and instructional video can be found at: <https://www.petmicrochiplookup.org/>



www.rcdas.org